Quala®
DENTAL PRODUCTS

Dental Amalgam Alloy

WARNING: CONTAINS MERCURY

May be harmful if vapors are inhaled.

Ingestion: May cause Neurotoxic Nephrotoxic effects.

Inhalation: May cause Bronchiolitis, Pneumonitis Pulmonary Edema.

Eyes & Skin: May cause redness and irritation to eyes and skin.

Contraindication: Do not use in persons with a known mercury allergy.

Precautions: Do not place the device in direct contact with other types of metals.

Use with adequate ventilation.

Store in a cool, well ventilated place.

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Mercury Complies to ISO 24234:2015

Keep Out of Reach of Children

CAUTION: Federal law restricts this device to sale by or on the order of a dentist

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QUALA CAPSULATED ALLOY

is a high copper content Ternary amalgam alloy. These alloys, as a group, are higher in strength and lower in creep than other types of amalgam alloys. Clinically, they have been shown to give desirable long lasting restorations. They handle well, develop strength rapidly and carve smoothly.

Directions for Use of Capsulated Alloy: This capsule is self-activating. It contains a premeasured container of mercury that activates and mixes with the premeasured quantity of amalgam alloy.

Trituration: This capsule system is recommended for use only in a high energy amalgamator. Variations in machines and voltage may require you to adjust actual trituration times slightly to your needs. If capsule does not activate at recommended setting, increase speed or time till activation occurs. As a general rule increased trituration makes the mix brighter, wetter and faster setting. Less trituration gives the mix more body and slows the setting speed.

WARNING

Ingestion: May cause Neurotoxic Nephrotoxic effects. In case of ingestion, seek medical attention immediately.

Inhalation: May cause Bronchiolitis, Pneumonitis Pulmonary Edema. In case of inhalation, remove to fresh air.

Seek medical attention.

Eyes & Skin: May cause redness and irritation to eyes and skin. In case of contact with eyes, wash thoroughly with water for 15 minutes holding upper and lower eyelids. Seek medical attention. In case of contact with skin, wash thoroughly with soap and water. Seek medical attention.

Acute Exposure: May cause sensitization dermatitis and possible visual disturbances.

WARNING:

Contains Mercury. Can cause birth defects and other reproductive harm.

www.P65Warnings.ca.gov

Keep Out of Reach of Children

CAUTION: Federal law restricts this device to sale by or on the order of a dentist.

INFORMATION FOR USE

"Dental amalgam has been demonstrated to be an effective restorative material that has benefits in terms of strength, marginal integrity, suitability for large occlusal surfaces, and durability. Dental amalgam also releases low levels of mercury vapor, a chemical that at high exposure levels is well-documented to cause neurological and renal adverse health effects. Mercury vapor concentrations are highest immediately after placement of and removal of dental amalgam but decline thereafter.

Clinical studies have not established a causal link between dental amalgam and adverse health effects in adults and children age six and older. In addition, two clinical trials in children aged six and older did not find neurological or renal injury associated with amalgam use.

The developing neurological systems in fetuses and young children may be more sensitive to the neurotoxic effects of mercury vapor. Very limited to no clinical information is available regarding long-term health outcomes in pregnant women and their developing fetuses, and children under the age of six, including infants who are breastfed.

The Agency for Toxic Substances and Disease Registry's (ATSDR) and the Environmental Protection Agency (EPA) have established levels of exposure for mercury vapor that are intended to be highly protective against adverse health effects, including for sensitive subpopulations such as pregnant women and their developing fetuses, breastfed infants, and children under age six. Exceeding these levels does not necessarily mean that they adverse effects will occur.

FDA has found that scientific studies using the most reliable methods have shown that dental amalgam exposes adults to amounts of elemental mercury vapor below or approximately equivalent to the protective levels of exposure identified by ASDR and EPA. Based on these findings and the clinical data, FDA has concluded that exposures to mercury vapor from dental amalgam do not put individuals age six and older at risk for mercury-associated adverse health effects.

Taking into account factors such as the number and size of teeth and respiratory volumes and rates, FDA estimates that the estimated daily dose of mercury in children under six with dental amalgams is lower than the estimated daily adult dose. The exposures to children would therefore be lower than the protective levels of exposure identified by ATSDR and EPA.

In addition, the estimated concentration of mercury in breast milk attributable to dental amalgam is an order of magnitude below the EPA protective reference dose for oral exposure to inorganic mercury. FDA has concluded that the existing data support a finding that infants are not at risk for adverse health effects from the breast milk of women exposed to mercury vapors from dental amalgam."

Compressive strength @ 1hour - High Cooper Alloy 108.2 MPa

Compressive strength @ 24 hours - High Cooper Alloy 445 MPa

Maximum Creep - High Cooper Alloy .098%

Dimensional change during hardening - High Cooper Alloy .018%

Particle size distribution (u) and shape - High Cooper Alloy -275 mesh, Irregular

Working time(s) (Eames test method) - High Cooper Alloy- Fast Set 5-6 minutes, High Cooper Alloy - Regular Set 6-7 minutes

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